PREMIUM-Dental-System / PREMIUM dental system

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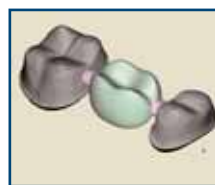
With the " Premium - system " for highly exact machining, particularly supplemental and optimised for the dental technology, developed a product, that in the CAD / CAM - Dental market sets a standard. The modular system is suitable for the machining of all materials and works with the open 3D-CAD/AM-Modul " isy CAM 3.0 ", which was automated with an application.

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The universal model (jaw) is digitised with the scanner, afterwards the single stumps.

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The digitised work is defined in the CAD modulation software (e.g. Links, connecting posts, position and size to the jawbone) By one mouse-click the data are stored and through the open interface as open STL data to the CAM module handed over. The computation parameters are put down for the individual materials and tools, and according to the material type are loaded automatically in the CAM module.

In the simulation the tool paths can graphically represented, and the complete machining is examined during the simulation.

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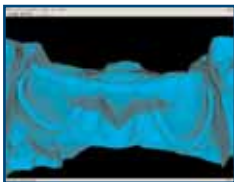
Expression of the comprehensive competence is also the XAWEX CAD/CAM production system for artificial dentures made of pure zircon oxide ceramics.


Zirconium oxide is a high efficiency ceramic. It has outstanding mechanic and physical characteristics. This material is undisputed the ideal material to produce metal-free crowns and bridge trestles and tooth-coloured dowel-superstructures.

Especially the high physical strength required expenses for investments in machinery for production of trestles out of zirconium oxide.

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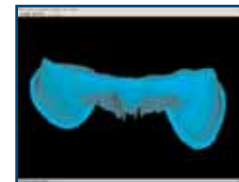First the gypsum model and the construction features are digitized with the integrated laser.

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With simple software tools the data are cut.

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The WOL CERAM® system is equipped with a heat exchanger, laser, ELC - dipping basins, micro motor, electrolyte and five magazines for fully automatic manufacturing.

The machine is able to scan and measure by means of a laser the stump of the tooth clamped in a rotary axis contactlessly. A pertinent software package processes the three-dimensional data developed thereby.

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The special advantage of this procedure is in the extremely accurate preparation of the WOL CERAM® cap, bridge or implant. The original stump is dipped into alumina or Zirconium and coated by the ELC procedure, and then a unique accurate interior fit develops. The Perfect Fit.

The desired layer strength is entered at the screen and the ELC CAM program settled fully automatic the production procedure. This manufacturing process is a multiple quicker and also connected with a substantial costing and material saving.